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**Intended use**

Dental splinting material short-term

**Extended purpose**

Thermoforming blank for the production of bleaching splints.

**Indications for use**

Suitable for the production of stable bleaching splints ending at the gingival margin and for situations with slight undercut areas. Bonds with acrylate.

**Contraindication**

A disinfection with all commercial materials do not use BioBleach® hart. Not desired side effects of this medical device are not to be expected in case of proper processing and application. However, immune reactions (e. g. allergies) or irritations principally cannot be excluded. Should you get to know non-desired side effects – also in doubtful cases – we appreciate in any case to be informed with a detailed description of the attendant circumstances and symptoms. We will follow each hint

**Target patient group**

Patients who need dental care

**Intended users**

Trained staff in dental field

**Processing**

BioBleach® hart will be delivered with an insulating foil. This extremely thin foil (0.07 mm) is thermoformed together with the BioBleach® hart on the model side and guarantees a decisively improved transparency of the splint. Thus an additional insulation of the model surface is no longer necessary. **If processed in a Vacformat, the insulation foil must be removed before thermoforming.**

The adjustment of the heating and cooling times can either be taken from the survey of thermoforming foils, or can be programmed directly into the unit with the help of the barcode on the BioBleach® hart packaging. Before thermo-forming the blank a space maintainer as a reservoir for the bleaching material has to be applied on the labial areas of the teeth.

a) With the light curing block out material Cenit L(☞D3815) the areas of the plaster model, which are supposed to be bleached, are evenly covered and afterwards get cured.

b) The whole model is covered with the thermo-forming blank Drufofen W 0.8 mm (☞D423481N). Then the part of the model to be bleached (incl. a slight overlapping of the cutting edges) is cut out of the blank by means of a pair of scissors and put back onto the model again.

**IMPORTANT:** In order to avoid any contact between the mucosa and the bleaching material the space maintainer should end approx. 1 mm above the gingiva margin.

**Disinfection and reuse**

A disinfection with all commercial materials is possible. BioBlach hart. used as a bleaching splint, can usually be reused maximum 4 times for 20 minutes each time.

**Note**

• While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.

**Serious incidents**

All serious incidents occurring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/ or patient is settled.

**Storage conditions**

At room temperature, i. e. 18–28 °C. Protect against humidity, always close foil bag when packs are in use.

**Lot number / Expiry date**

The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

**Disposal**

Disposal of contents/container in accordance with local/regional/national/international regulations.

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This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible

use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.

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