
Bedienungsanleitung · Working instructions
Mode d'emploi · Instrucciones de trabajo
Istruzioni d'uso · Gebruiksaanwijzing

Dosper M/5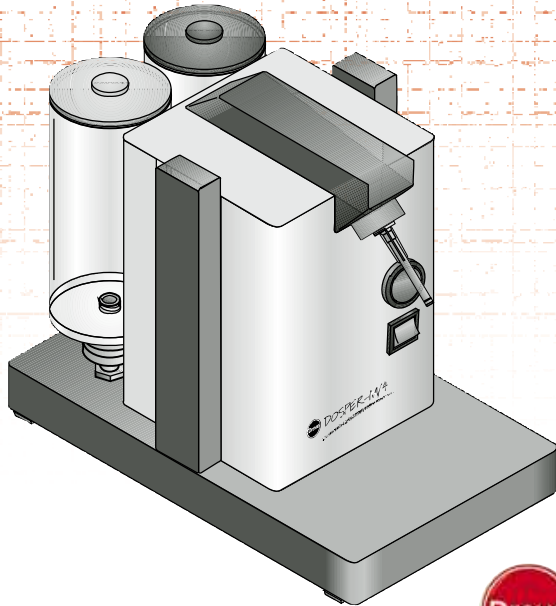

Dreve · Max-Planck-Straße 31 · 59423 Unna/Germany

Dosper M/5

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Technical data

H x W x D: 330 x 245 x 370 mm
 Weight: 12.5 kg
 Voltage: 230 V/50 Hz,
 220 V/60 Hz, 115 V/60 Hz
 (Voltage/frequency see type plate)

Power input: 250 W
 Housing: Robust metal housing,
 coated with powder
 Gearmotor with super-heated
 protection

EU-Declaration of conformity

According to the EU-guideline machines 89/392/EWG, appendix II A, EU-guideline for electromagnetic compatibility EMC 89/396/EWG and the EU-guideline for electrical appliances for low tension 73/23/EWG

We herewith declare that the machine described below is in its conception and design and in the shape delivered by us in accordance with the fundamental requirement for safety and health as prescribed in the applicable EU-guidelines. In the case of any change or modification of the machine not authorised by us this declaration becomes invalid.

Designation of the machine:

Dosper M/5

Type of machine:

Dosing and mixing unit for silicone masses

Applicable EU-guidelines:

89/392/EWG	EU-guideline machines
89/336/EWG	EU-guideline for electromagnetic compatibility EMC
73/23/EWG	EU-guideline for electrical appliances for low tension

Applied harmonised standards:

EN 292-1	Safety of machinery; basic concepts, general principles for design; part 1: basic terminology, methodology
EN 292-2	Safety of machinery; basic concepts, general principles for design; part 2: technical principles and specifications
EN 50081-1	Expert basic standard interference sending, living area
EN 50082-2	Expert basic standard interference resistance, living area
EN 60204 Part 1	Safety of machinery, electrical equipment of machines; Part 1: general requirements
EN 61010 Part 1	Safety requirements for electrical equipment for measurement, control and laboratory use; Part 1: general requirements

Dreve
 Max-Planck-Str. 31, 59423 Unna/Germany
 Tel.: +49 2303 8807-0
 Fax: +49 2303 82909
 Homepage: www.dreve.com
 Email: info@dreve.de

deutsch

english

français

español

italiano

nederlands

Dosper M/5

Safety instructions

Attention! Read the following notes before connecting and putting the unit into service. The operating service and the function of the unit can only be guaranteed for, if the general safety and accident prevention instructions of the legislation as well as the safety instructions of the working instruction are observed.

1. This unit must only be used according to the present working instructions. We cannot be held liable for damages which have been caused by improper treatment respectively faulty manners of operation.
2. The indicated voltage on the type plate has to meet the voltage of the power source.
3. Connect the unit only to a socket with protective conductor. Never touch the plug with wet hands.
4. The unit has to be placed onto a stable and even surface.
5. Do not intrude any objects into the unit.
6. Only those appliances and spare parts may be used which are released by the manufacturer.

We are not responsible for damages caused by the use of outside parts.

7. Do not lock the operation button of the unit.
8. The unit has to be examined before operation in view of proper condition and general safety. If the unit is not in proper condition, it must not be used and has to be marked accordingly.
9. Before cleaning and maintenance of the unit or the changing of parts the plug has to be disconnected.
10. Plates and labels have to be maintained in a well legible condition and must not be removed.
11. The opening of the unit and repairs are only to be carried out by accordingly trained experts.

Description of the unit

The Dosper M/5 is an automatic mixing and dosing unit for thin-flowing addition-vulcanising two-component duplication silicones. At the touch of a button the components are mixed in a ratio of 1:1 and dosed

for duplication. The quick-coupling allows the use of the 1 kg original packaging drums, refillable material reservoirs or the 6 kg resp. 25 kg large cubitainers.

Putting into service

Ensure that the indicated voltage on the type plate meets the voltage of the power source.

Plug the power cable into the socket of the unit and create a connection with the euro-plug.

Operation

Only use high-quality duplication silicones. Inhomogeneous viscosities and filler distributions in the single components can lead to a difference of the ideal 1:1 ratio due to the abrasion in the pump housing. This can cause time-consuming and cost-intensive repairs. We recommend to use Dublisisil silicones.

Preparation of the silicone containers

1 kg bottles, material reservoirs or cubitainers can be connected with the Dosper M/5.

Bottles: Open the bottles with the components A and B and apply bottle adapter.

Attention: Coloured bottle adapter onto the bottle filled with coloured silicone mass - component B.

Material reservoir: The colour-marked transparent containers which have already been equipped with a quick-coupling are optionally available. The coloured silicone component has to be filled into the colour-marked reservoir.


Cubitainer: Unscrew the caps of the canister and fit colour-marked connection hoses.

Attention: Keep the two silicone components always separated - do not interchange connections! In case the two components have been mixed the silicone cures in the unit which makes considerable repairs in our plant in Unna necessary.

Filling of the unit


The Dosper M/5 is delivered ex-works with testing liquid in the system. When putting the unit into service this liquid is produced by touching the button until the duplication mass reaches the hoses.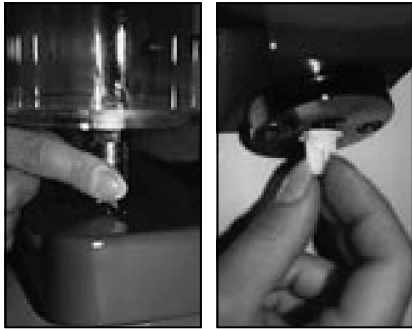

Remove the lock caps from the coupling plugs (no. 19). Press the quick-coupling/bottle adapter of the silicone containers onto the coupling plugs of the unit and let them snap hearably.

Attention! The components A and B have to be connected according to the printing on the unit and the applied colour marking. Cut an opening of

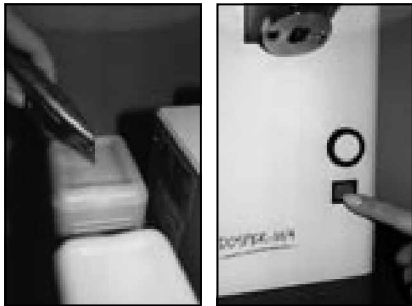approx. 0,5 cm² into the bottoms of the cubitainers in order to let air come in when producing. Remove protection cap from mixing head (no. 4) and do not reuse. Turn on the unit at the operation button. Put a reception bowl under the mixing head and press push-button (no. 2). Keep the push-button pressed until the test silicone has been produced completely

and both components come out homogeneously at the mixing head. Fix mixing canula at the mixing head (no. 4) by means of turning it to the right by 90°. The turning of the mixing canula can be eased by means of the enclosed star-turn handle (Item-no. 1565E).

The Dosper M/5 is now ready to operate. Put duplication flask under the opening of the mixing canula and press push-button (no. 2). As long as the material is not cured in the mixing canula (approx. 4 minutes) several flasks can be filled in succession.

After having finished the filling process do not remove the mixing canula. Do only remove the mixing canula before the next duplication by means of a 90° turn to the right.

Attention: Never produce against a mixing canula with cured material!

Care and maintenance

Disconnect the unit from mains supply before care and maintenance! Cleaning is effected with a mild cleaning liquid.

Change of bottles/cubitainers

Remove the quick-coupling/bottle adapter from the unit by means of tearing upwards the lock-up. Screw quick-coupling/bottle adapter onto the new bottles/cubitainers and connect as described above. Observe the correct coordination of the components according to the colour.

Observe the contents of the containers at any time! They must not be emptied completely, as otherwise air gets into the system and the unit has to be bled under great effort. A minimal filling of 2-3 cm has to remain in the containers.

General notes

Repairs on the Dosper M/5 may only be executed by experts. Disconnect the unit from mains supply at any rate. Drevo company is only responsible for the safety, reliability and performance of the unit, if

- the unit is exclusively used for the corresponding field of application.
- operation is only carried out by accordingly trained personnel.

Wipe away silicone remainders from the unit. Keep coupling plug and mixing head clean. The cleaning liquid must not get into the unit.

- the unit is operated in accordance with these working instructions.
- extensions, new adjustments, changes or repairs are executed by persons authorised by the company.

As we are currently developing our products, we reserve the right for technical changes.

Trouble shooting


Problem	Cause	Aid
Unit does not produce.	Mixing head is clogged.	Clean the mixing head carefully by means of the enclosed cleaning spirals. Attention: Use a different spiral for each drilling, as otherwise the two components would mix and cure.
Cleaning of the mixing head		
		
Unit does not produce.	Cured material in old mixing canula.	Assemble new mixing canula.
Unit does not produce.	The components A and B have been interchanged; the silicone has cured in the system of the unit.	The unit has to be dismantled completely and cleaned by an expert.

Exchange of the miniature fuses

Problem	Cause	Aid
Unit does not produce.	The filling of the silicone containers has fallen below the minimum filling; the system has sucked air.	Unscrew mixing head flange (no. 27) with a 4 mm hexagon spanner, take out mixing head (no. 3) to the front. Loosen the union nuts (no. 8a), spanner of 10 mm, from the hoses and tear off hoses. Refill silicone material and wait for approx. 20 minutes until the silicone has flown into the pipe. Put reception bowl under the opening of the hoses, switch unit on and press button until both components have come out of the hoses free of air bubbles. Reassemble unit in reversed succession.
Unit does not produce; lamp of the operating button does not light up.	Miniature fuse is defect.	Examine miniature fuse and exchange, if necessary.
Unit does not produce in a mixing ratio of 1:1.	The silicone components do not have the same viscosity.	Use material recommended by the manufacturer (a deviation of about 5 % can be tolerated).
Unit does not produce in a mixing ratio of 1:1.	The tooth wheels in the pump are worn unequally.	The unit has to be overhauled by an expert.

Ventilation of the production system

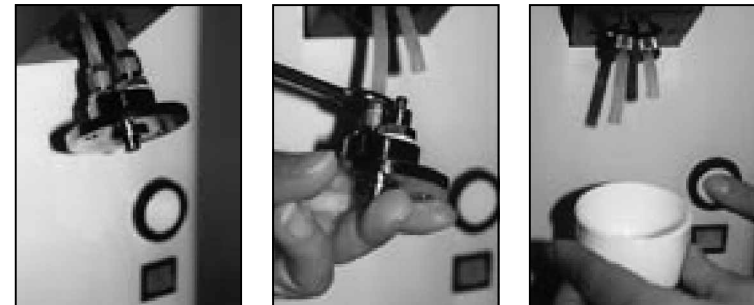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Garantie

The right of the Purchaser to bring actions arising out of defects shall in all cases be in lapse 2 years after the date of delivery. The guarantee obligation is only valid for material or production defects. It does not refer to natural wear and tear nor to damage occurring after the transfer of risk due to improper or unsuitable handling, excessive stressing, unsuitable operating material, and chemical, electrochemical or electrical influences of a nature not provided for according to the contract. On improper alterations or repairs or maintenance carried out by the Purchaser or third parties the guarantee obligations of the Supplier and his liability for consequences caused thereby are rendered void.

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