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Tandheelkundig lepelmateriaal kortstondig

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Intended use

Dental splinting material short-term

Extended purpose

Thermoforming blank for the production of bleaching and medical splints.

Indication for use

BioBleach® soft was especially designed as a splint material for home-bleaching. Also after the thermo-forming process the special formulation of the 1.0 mm strong EVA-material offers a consistent soft and stable splint. The usage for several bleaching processes is possible without any problems.

Processing instruction

The detaching of the splint from the cooled down thermo-forming blank is effected by means of a scissors or a hot scalpel. Due to the good elasticity of the splint material the marginal limit of the blank can outreach the gingival margin. This extends the vent edge of the splint and therefore guarantees the safe position of the bleaching material in the reservoir. In order to avoid irritations of the gingiva and the cheek the cut edges of the thermoforming splint should carefully be rounded off with a rotating instrument (e. g. UltraTrim light) and smoothed.

Disinfection and reuse

A disinfection with all commercial materials is possible. If BioBleach® soft is used as a medical splint, it can usually be reused maximum 10 times for 20 minutes each time. As a bleachng splint only 4 times for 20 minutes each time.

Target patient group

Patients who need dental care.

Intended users

Trained staff in dental field

Preparative measures

Before thermo-forming, the blank a space maintainer as a reservoir for the bleaching material has to be applied on the labial areas of the teeth. For this exist two possibilities:

a) With a light-curing easy flowing but thixotropic resin material (Cenit L) the areas at the plaster model which are supposed to be bleached are consistently covered with a brush and afterwards get cured.

or

b) The whole model is covered with the thermo-forming blank Drufolen W 0.8 mm. Then, the part of the model to be bleached (incl. a slightly overlapping of the cutting edges) is cut out of the blank by means of a scissors and put back onto the model again.

IMPORTANT: In order to avoid any contact between the mucosa and the bleaching material the space maintainer should end approx. 1 mm in front of the gingival margin.

Disposal

Disposal of contents/container in accordance with local/regional/national/international regulations.

Contraindication

This product was developed for dental use and has to be used according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before

Processing instruction

Insert the BioBleach® soft blank in the thermo-forming unit and heat it. The right plastification is achieved when the blank sags for approx. 10–15 mm.

After having achieved the right plastification drag the blank directly over the model and form it with vacuum or pressure.

IMPORTANT: In order to realise a good and stress free splint the pressure/ vacuum has to be adhered for approx. 5 min. until the complete heat exchange of the blank.

Processing instruction

The detaching of the splint from the cooled down thermo-forming blank is effected by means of a scissors or a hot scalpel. Due to the good elasticity of the splint material the marginal limit of the blank can outreach the gingival margin. This extends the vent edge of the splint and therefore guarantees the safe position of the bleaching material in the reservoir. In order to avoid irritations of the gingiva and the cheek the cut edges of the thermoforming splint should carefully be rounded off with a rotating instrument (e. g. UltraTrim light) and smoothed.

Disinfection and reuse

A disinfection with all commercial materials is possible. If BioBleach® soft is used as a medical splint, it can usually be reused maximum 10 times for 20 minutes each time. As a bleachng splint only 4 times for 20 minutes each time.

Note

• While working out on the material, it is recommended to use an exhaust system and protective glass or goggles as well as a face mask.

Serious incidents

All serious incidents occuring in relation to the product shall be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is settled.

Storage conditions

At room temperature, i. e. 18–28 °C.

Lot number / Expiry date

The lot number and the expiry date are indicated on each packaging. In case of claims, please always indicate the lot number of the product. Do not use the product after the expiry date.

Disposal

Disposal of contents/container in accordance with local/regional/national/international regulations.

Contraindication

This product was developed for dental use and has to be used according to its purpose, especially in case the purpose is not mentioned in the instructions.

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