### Indication

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## Indications

- for total and partial dentures
- for the completion of models cast dentures, total and partial relinings, immediate dentures, additions and repairs.

Colour stable, auto-polymerising resin on the basis of methylmethacrylate in the form of polymer and monomer. Barbiturate acid curing free of cadmium. Catalyst system free of tertiary amine.

Shrinkage: 5.7 %  
Rest monomer contents:  
12 mg / g after 48 hours in water of 37 °C  
6 mg / g after 28 days in water of 37 °C

## Mixing ratio

10 P. (Weight) Polymer / 7 P. (Weight) Monomer (if used as pouring resin).  
10 P. (Weight) Polymer / 4.5–5 P. (Weight) Monomer (if used as repairing material and for the completion of model cast dentures).

**Important!** The ideal processing temperature is 23 °C. Low temperature extends, high temperature reduces the processing time.

## Castdon Monomer

### Hazard statements

H225 Highly flammable liquid and vapour. H315 Causes skin irritation. H317 May cause an allergic skin reaction. H335 May cause respiratory irritation.

### Precautionary statements

P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P241 Use explosion-proof electrical/ventilating / lighting / equip-

## Processing

Mixing: 0.5 min.  
Pourability: 2–3 min.  
Monomer can be stored in the refrigerator.  
Cooler temperatures will prolong the pouring time.

Stability: after 5–7 min.  
due to prolongation of the plastic phase the material is mouldable for a long time for model cast technique

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## Contains

Methylmethacrylate, Butanone, Dibenzoyl peroxide

**Attention!** Because of risks, the material has to be cured. Avoid skin contact with unpolymerised material and do not breathe monomer vapours. Consider the safety data sheets, and supply the dentist with the above information.

**Keep away from children!**  
**Only for dental use!**

ment. P261 Avoid breathing dust / fume / gas / mist / vapours / spray. P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water / shower. P405 Store locked up. P501 Dispose of contents / container in accordance with local / regional / national / international regulations.

## Storage conditions

Store materials tightly closed at room temperature, i. e. 10 °C–25 °C.  
**Durability:** see date of expiry.  
**Castdon** is not to be used when expiry date has passed.


## Disposal

Polymer can be disposed in garbage, monomer has to be disposed as toxic waste.

## Note

The polymerisation reaction of **Castdon** is exotherm and should therefore only be carried out extra-orally. **Castdon** has been clinically well-proven for many years.

This product was developed for dental use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. Furthermore the user is bound to check the product according to suitability and possible use in self dependence before using it subject to its purpose, especially in case the purpose is not mentioned in the instructions.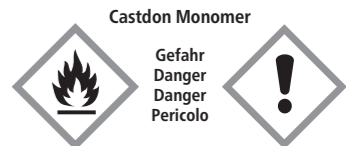

Castdon Monomer

Gefahr  
Danger  
Danger  
Pericolo