English

English

Working Instructions

MaxiFresh™

Fields of application:

The **MaxiFresh™** cleaning tablet is used for the outer cleaning and care of dental splints and sport mouthguards.

Frequency of use:

We recommend daily cleaning. This will prolong the durability of your top-quality dental splint and safeguard an impeccable hygiene and function.

Composition:

The cleaning tablet contains pentapotassium bis (peroxymonosulphate)bis(sulphate) disodium carbonate, compound with hydrogenperoxide (2:3), Anionic surfactant.

Application:

Put the cleaning tablet into the **MaxiFresh™** bowl filled with 150 ml of water with a temperature of 37 °C.

Please note: In cooler water the cleaning tablet might not dilute completely. After the dilution of the tablet, the reaction time should be at least 15 minutes, the maximum should be 60 minutes.

When reaction time has passed take the dental splint out of the **MaxiFresh™** bowl, and – with the help of the insert – rinse thoroughly with clear water. Then dry the surface.

Keep away from children! Only for dental use!

Storage conditions:

At room temperature, i.e. 18°C–28°C

Durability: see date of expiry

MaxiFresh™ is not to be used when expiry date has passed.

Form of delivery:

Item no. D-4100

**24 MaxiFresh™
cleaning tablets incl.
MaxiFresh cleaning bowl**

Item no. D-4101

**24 MaxiFresh™
cleaning tablets Refill Set**

Hazard-determining components of labelling:
pentapotassium bis(peroxymonosulphate)bis(sulphate), citric acid, disodium carbonate, compound with hydrogenperoxide (2:3), Anionic surfactant

Hazard statements:

H302 Harmful if swallowed.
H314 Causes severe skin burns and eye damage.

Precautionary statements:

P101 If medical advice is needed, have product container or label at hand.

P102 Keep out of reach of children.

P103 Read label before use.

P260 Do not breathe dust/fume/gas/mist/vapours/spray.

P303+P361+P353

IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

P305+P351+P338 IF IN EYES:

Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

P310 Immediately call a POISON CENTER or doctor/physician.

P405 Store locked up.

P501 Dispose of contents/container in accordance with local/regional/national/international regulations.

This product was developed for dental lab use and has to be applied according to the instructions. In the case of damages resulting from ulterior or improper use, the manufacturer assumes no liability. The user is generally obligated to check the product before use. Especially if the purpose is not mentioned in this instructions.

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